Inhibit Adapter

Installation Instructions

EN

1. Remove the cover from the AEM® Monitor.
2. Cut the inhibit adapter cable at the strain relief on the inside of the unit next to the sheet metal and pull the inhibit adapter cable out from the front.
3. Disconnect the white locking connector that attaches the inhibit adapter cable to the ESM Main board.
4. Mark the color of the wire from the inhibit adapter onto the locking connector.
5. With a knife or small screwdriver, push the metal part of the contacts down on the white connector. Push the contact down and toward the wire. Pull the wire after pushing the contact back. The contact should pull out of the white connector.

NOTE

- Look at the metal contact while removing – the correct orientation of the contact is needed to place the new one.
6. Insert the inhibit adapter cable from the new inhibit adapter through the hole in the front panel. A pair of pliers may be used to pull the cable through. Pull only on the strain relief portion of the cable.

NOTE

- Squeezing and holding the strain relief will compress the material and make it easier to pull through the front panel. The use of alcohol on the strain relief may also help.
7. Insert the metal contacts into the white connector in the correct locations. Gently pull the wires after all of the contacts have been inserted to insure they are locked into position.
 8. Attach the white connector to the PC Board.
 9. Reattach the cover onto the top of the unit.

Verify that the AEM Monitor functions properly per Section 3 of the AEM Monitor Operator/Service Manual.

Perform normal electrical safety tests as required per hospital protocols.

For questions, contact the Encision Service Department.

ENCISION® and AEM® are registered trademarks of ENCISION Inc.

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© Copyright 2010 Encision Inc.
ENCISION® en AEM® zijn gedeponeerde handelsmerken van ENCISION Inc.