





Watch the AEM EndoShield video now!

## Quick Set-Up

- 1 Connect AEM EndoShield to the electrosurgical generator.
- 2 Then connect the patient return electrode to the front of the AEM EndoShield.



Illuminates RED until an AEM instrument is connected.

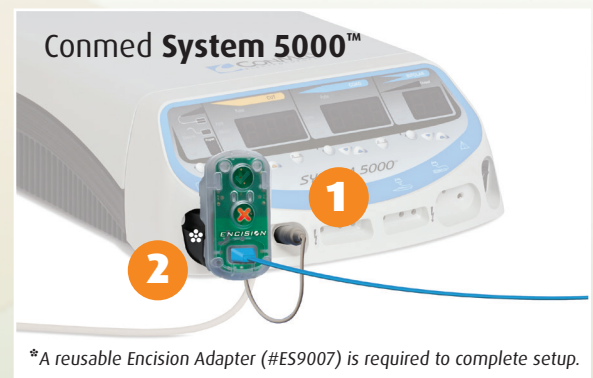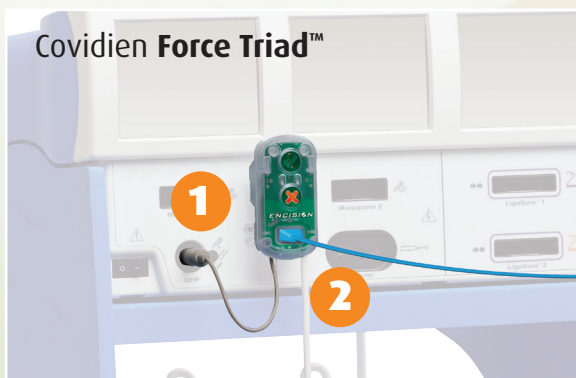


\*A reusable Encision Adapter (#ES9007) is required to complete setup.


- 3 Connect an AEM foot-control instrument to the AEM EndoShield.



Illuminates GREEN when the AEM EndoShield Burn Protection System is ready for surgery.

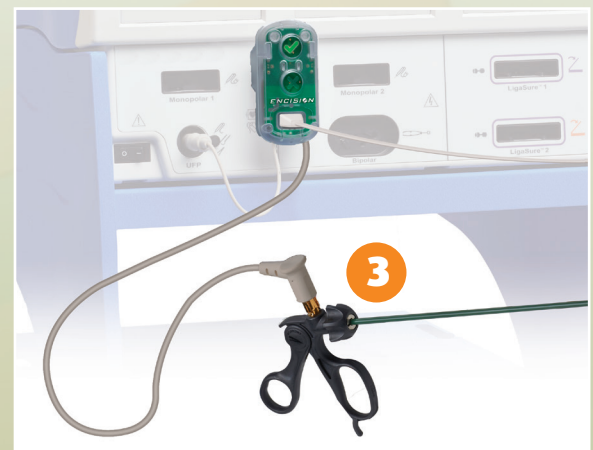

## Troubleshooting

If the  illuminates during surgery and power is not delivered to the AEM instrument, follow these simple steps:

1. Check the setup of the AEM Burn Protection System as described above.

If this does not fix the issue, it may be due to the AEM system detecting an insulation failure in the instrument or EndoShield. Therefore the AEM instrument may need to be replaced.

2. Replace the AEM instrument. If the problem persists, also replace the AEM EndoShield.

